

Tourniquet Cuff Covers - Padded

Instructions for Use

Cuff Covers - Small: 40305

Cuff Covers - Large: 40315



1. Introduction

This guide refers to the following products;

40305 Cuff Covers – Small

40315 Cuff Covers – Large

These products are manufactured by Anetic Aid Ltd. They are CE marked against the requirements of the Medical Device Regulation 2017/745, and are classified as a Class 1 medical device.



Anetic Aid Ltd.
44 New Lane
Havant, Hampshire,
PO9 2NF



EU-UK Authorised Representative Ltd.
Unit 591, Moat House, 54 Bloomfield Avenue,
Belfast, BT5 5AD, Northern Ireland

Our representatives are available for on-site consultation or training on any of these products and our head office team will be pleased to answer any queries you may have.

2. Product Details

Anetic Aid Tourniquet Equipment is highly acclaimed for its reliability and user friendly designs. Widely used across the UK, these devices will be found ideal for Theatre, A&E and Pain Relief Clinics.


The devices intended purpose is as padding between a patients limb and a tourniquet cuff, and as a barrier to prevent ingress of fluid to the area between the patients limb and the tourniquet cuff inflation chamber.

- The device is supplied latex free, non-sterile and as single use.
- The product is supplied in two sizes, small and large.

3. Instructions for Use

Product should only be used, for its intended purpose, by suitably trained personnel. Familiarise use of product on an appropriate volunteer, prior to introducing to clinical use.

Before each use, inspect product for signs of deterioration or damage; discard any device found to be defective, do not attempt to repair.

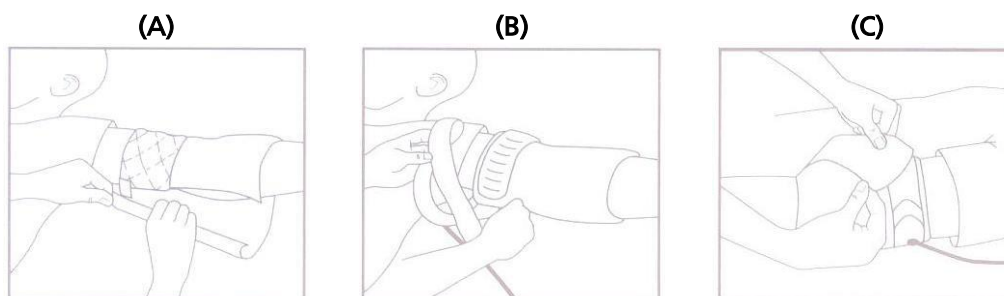

Applying a cover;

1. Remove the backing from the bottom seal tape.
2. **(A)** Wrap the padded portion of cover on limb at desired location of tourniquet, ensuring the bottom seal tape is applied in a smooth, kink free manner.
3. Compress by hand over the whole area of the bottom seal tape to ensure a complete seal is made.
4. **(B)** Apply and secure tourniquet cuff over padded area of cover.

5. (C) Pull bottom portion of cover up over tourniquet cuff. Tuck in at the top to completely cover tourniquet.
6. On completion the device should be removed and discarded according to site policy

4. Care & Maintenance

- Store in the packaging provided in a cool dry place away from radiators and other heat sources.
- The product will be adversely affected and its life expectancy reduced if the following cautions are not observed;

CAUTIONS:

- Do not use if found damaged or wet.
 - Do not reuse, single use only.
 - Do not allow contact with bleaching disinfectant solutions, organic solvents or oil based products.
 - Do not expose any part of the product to excessive heat.
 - Do not press with hard or sharp objects.
-

For more information on any of our products or service contracts, call:
sales@aneticaid.com **+44 (0)1943 878647** aneticaid.com



DESIGN & INNOVATION



MADE IN THE UK



GLOBAL DISTRIBUTION

Innovative **Medical** Technology - **Practically Applied**



Anetic Aid Ltd., 44 New Lane, Havant, PO9 2NF, UK

