

Quick Start Guide for //////**DRIVE** ASSIST

Innovative medical technology – practically applied



AneticAid

This 'Quick Start Guide' is designed to help you use the //////**DRIVE** assist function in nine simple steps. However, it is advisable to read the full 'Instructions for Use' thoroughly before operating the trolley.



1



Engage the //////**DRIVE** wheel; see section 10 of the 'Instructions for Use'.

2



Press the ON/OFF button; a short beep will sound. Do not touch the //////**DRIVE** handles immediately after pressing the ON/OFF button.

3



The ON/OFF light will flash green for 1-3 seconds as //////**DRIVE** calibrates. The ON/OFF light will turn solid green when calibration is successful, and a short beep will sound. //////**DRIVE** is now active.

4



Disengage the brakes; see section 9 of the 'Instructions for Use'.

continued overleaf...

5



Driving: Push positively on the **//////DRIVE** handles and the trolley will begin to move forwards. Push harder and the trolley will move more quickly. Pull back on the **//////DRIVE** handles to slow, stop, or reverse, the trolley.

6



Driving up a slope: Push the **//////DRIVE** handles fully forward, and walk at the pace of the trolley. Pushing harder will result in unnecessary physical exertion, and the speed of travel will not be increased.

7



Driving down a slope: As you approach a downhill slope, pull back gently on the **//////DRIVE** handles to decrease the trolley speed. As you walk down the slope, continue to pull back gently on the **//////DRIVE** handles to descend the slope under control.

8



After 3 minutes of inactivity, the **//////DRIVE** handles will deactivate; this is indicated by 3 rapid pulses of the ON/OFF light. The **//////DRIVE** handles are reactivated by a short press of the ON/OFF button. After 30 minutes of inactivity, the system will switch off.

9



If at any stage, one or more of the amber lights come on, refer to full 'Instructions for Use'.