

Extended Arm Table - Lightweight

Instructions for Use

Code 10233



1. Introduction

These instructions are intended to assist you with the use and maintenance of the extended arm table. It is important that these instructions are read thoroughly before using the equipment. The device will be adversely affected and its life expectancy reduced if the following instructions are not observed.

NOTE: If the extended arm table is damaged or faulty it **must** be taken out of use with immediate effect and the fault reported to Anetic Aid, your authorised dealer or maintenance department. The extended arm table **must not** be used until the damage or fault has been repaired.

1.1. Warnings & Cautions

Various warnings and cautions are made throughout these operating instructions.



A **WARNING** is given when the personal safety of the patient or user may be affected and when disregarding this information could result in injury.



A **CAUTION** is given when special instructions must be followed. Disregarding this information could result in permanent damage being caused to the trolley.

1.2. Intended Use & Contraindications

This product is intended for use within a theatre environment when special access is required to the arm or hand of the patient.

1.3. Device Classification

The device referenced in this document is CE marked and has been classified as a Class 1 Medical Device under the scope of both the Medical Devices Directive 93/42/EEC and the Medical Device Regulation 2017/745.



EU-UK Authorised Representative Ltd.
Unit 591, Moat House, 54 Bloomfield Avenue,
Belfast, BT5 5AD, Northern Ireland



OZG OneZurich Group Ltd.,
Mülibodenstrasse 3, 8172 Niederglatt ZH,
Switzerland

1.4. Putting the Extended Arm Table – Light Weight into Service

It is important that the extended arm table is cleaned and disinfected before it is put into service. Use this manual to check all the functions and refer to the 'Cleaning and Disinfecting' instructions.

2. Product Specifications

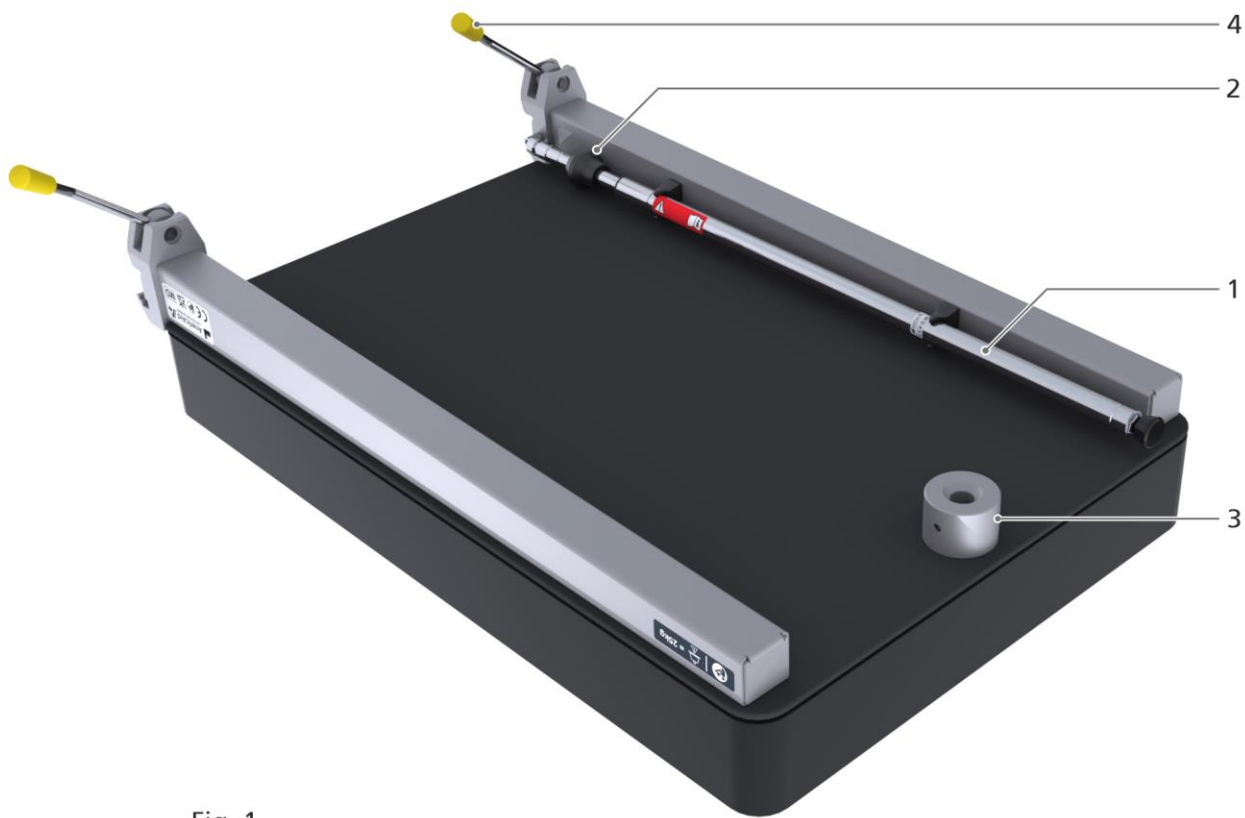

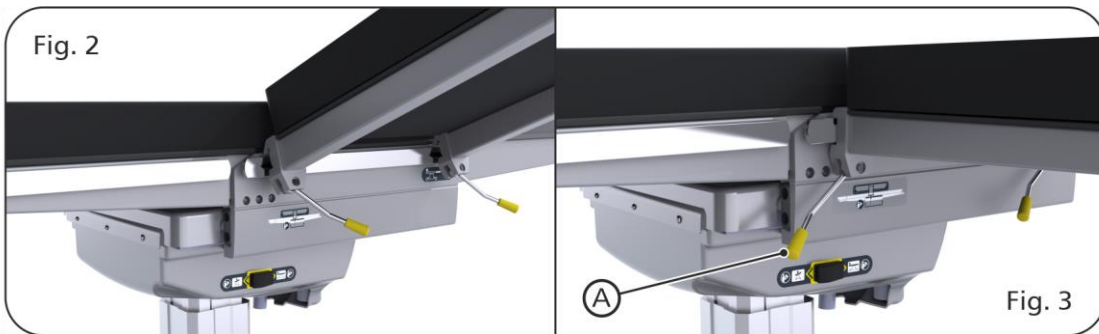


Fig. 1

Key to Fig. 1	
1.	Adjustable Support Leg
2.	Adjustable Support Leg Locking Mechanism
3.	Location Boss for Support Leg
4.	Locking Clamp Handles
Dimensions:	
Product dimensions	L685mm x W450mm x D165mm
Product weight	6kg
Weight Limit:	
Safe working load	25kg (55lbs)
	WARNING: Exceeding the maximum specified weight limits could result in failure of the extended arm table and injury to the user.

3. Direct-on Attachment

The extended arm table attaches 'direct-on' to the trolley by hooking onto the side bar, see Fig.2. It is designed to fit UK and EU side bar. To secure the arm table, rotate the locking clamp handles 90° in a clockwise direction, Fig.3 (A).



WARNING: It is recommended both locking clamp handles are in the locked position before using the product to prevent any possible movement during surgery.

4. Using the Adjustable Support Leg

The adjustable leg is stored in the clips on the underside of the arm table and can be used to add extra stability during surgery (item 1, Fig.1).

First, insert the leg into the location boss until it 'clicks' into place (item 3, Fig.1). Grasp the adjustable leg just below the locking mechanism (item 2, Fig.1), and using your thumb lift the mechanism to release the lock. In a controlled manner lower the leg until it meets the floor and release the lock. To remove the leg from the location boss, grip the leg by the locking mechanism and pull firmly; failure to pull from the locking mechanism will cause the leg to extend.



WARNING: The adjustable support leg must be removed or reduced in length before lowering the table, failure to do so will result in damage to the arm table, the trolley and possible injury to the patient and user.

5. Cleaning and Disinfecting

It is recommended that only CE marked cleaning products are used in the cleaning of this device. Cleaning and Disinfection should be carried out by hand only.

Clean the light weight extended arm table with warm water and neutral detergent and dry the surfaces thoroughly using a soft cloth. Suitable disinfectants are: quaternary ammonium compounds, isopropyl alcohol & chlorine bleach up to 1% (10,000 ppm). Apply disinfectant by cloth, spray or disinfectant wipe. Following disinfection, wash off the all surfaces with clean warm water and dry thoroughly using a soft cloth.

The product will be adversely affected and its life expectancy reduced if the following cautions are not observed;

CAUTION:



- Do not steam clean or jet wash any area of this device.
- Do not soak or immerse the device.
- Do not use concentrated bleaching disinfectant solutions, organic solvent or abrasive powders in the cleaning or disinfection of this product.
- Dilute all disinfectants in accordance with the manufacturer's guidelines.
- Disinfectant products are corrosive in nature; failure to properly wash and dry the product surface could leave a corrosive residue which may cause damage to the product.

6. Product Maintenance

The extended arm table does not require servicing. However, before use, visually inspect the product for any loose or damaged parts. Every six months we recommend the arm table is inspected by a competent person, i.e. an Anetic Aid service engineer, to ensure that it is safe to use; contact Anetic Aid for further details.

NOTE: If the light weight extended arm table is damaged or faulty it **must** be taken out of use with immediate effect and the fault reported to Anetic Aid, your authorised dealer or maintenance department; it **must not** be used until the damage or fault has been repaired.



CAUTION: In line with the MHRA Device Bulletin DB2006(5), maintenance work should only be conducted by suitably trained personnel following manufacturer's guidelines.

7. Product Warranty

The product, when new, is guaranteed to be free from defects in materials and workmanship and to perform in accordance with the manufacturer's specification for a period of five years from the date of purchase from Anetic Aid Ltd or their approved Distributor. Anetic Aid Ltd will repair or replace, at their discretion, any components found to be defective or at variance with the manufacturer's specification within this time at no cost to the purchaser. The warranty will take effect from the date of purchase, subject to the purchaser registering the product with Anetic Aid to confirm its receipt, installation date and product details.

The warranty does not provide cover for breakage or failure due to tampering, misuse, neglect, accidents, modifications or shipping. The warranty is also void if the product is not used in accordance with the manufacturer's instructions or is repaired during the warranty period by any persons other than Anetic Aid or its appointed agent. No other expressed or implied warranty is given.

For details of our extended warranty packages please contact Anetic Aid or your authorised dealer.

8. Label Identification

The following list is a description of all the labels used on the trolley;

Safe Working Load; indicates the safe working load of light weight arm table is 25kg (55lbs).



WARNING label; informing user to read the instructions for use document.



The Anetic Aid branded operating table accessory screen print with information symbols; CE mark, refer to instructions for use, UKCA, latex free.



This page has been left blank.

For more information on any of our products or service contracts, call:
sales@aneticaid.com **+44 (0)1943 878647** aneticaid.com



DESIGN & INNOVATION



MADE IN THE UK



GLOBAL DISTRIBUTION

Anetic Aid Ltd., 44 New Lane, Havant, PO9 2NF, UK

Document No. 992042. Issue 7 – 24.10.22

